### **(GB)** BREAM, BREAM KIDS, HYAMS and HYAMS KIDS cotton sewn mask

Respiratory mask suitable for protection against coughing droplets with laces (4 pcs) to tie the mask behind the head. The mask is reusable after disinfection by boiling and steam ironing.

**Material:** Cotton, prewashed

**Size:** Universal

**Color:** different colors

**Packaging:** 1 pcs / bag, 700 pcs/ box

**Storage:** dry, shady areas

#### Instructions for proper wearing of the mask

Wash your hands thoroughly before putting on and after removing the mask.

First put the veil on the chin and nose, then tie the top to the bottom with the laces behind the head, the lower part under the ear, the upper part above the ear.

**IMPORTANT:** Read this manual before use. Save this manual for future reference.

#### Disinfection

The mask can be used repeatedly after boiling and ironing. They must be submerged in water and boiled for at least five minutes. Allow the mask to cool in water after boiling. After drying, iron the mask with steam.

Washing masks - preferably choose a program with hot water - the minimum temperature is 60 ° C and the program with this temperature should last at least 30 minutes. After removing masks from the machine, dry them in a tumble dryer or freely and then iron with steam.

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